QUAD WALKING CANE

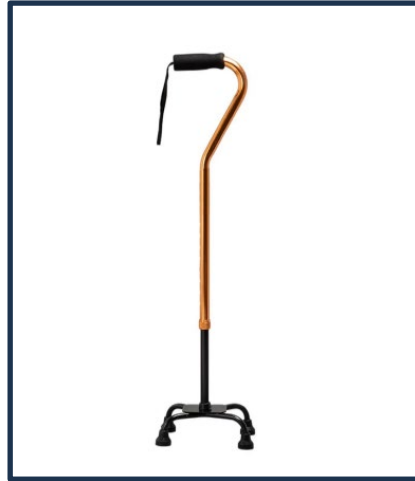
Item No: R288 / 75696

Instructions for Use

v001: 21/01/2026

Please read and retain these instructions for future reference

ENGLISH INSTRUCTIONS

INSTRUCTIONS FOR USE:

1. Before use, adjust the walking cane to a suitable height.
Adjustment method: Press the spring button with your finger, adjust it up or down to the appropriate height, then tighten the nut to secure it. Ensure it is firmly fixed before use.
2. Before use, check if the handle is loose or has burrs to avoid affecting safe use.
3. Before use, check if the walking cane's anti-slip foot pads are securely and stably attached. If worn, replace them promptly.
Check if the spring ball is flexible. Be especially careful when using it on slippery surfaces.
4. walking canes are walking aids; please do not use them as support for anything other than walking.
5. The user's weight should not exceed 100kg. It is not suitable for mountaineering or prolonged weight-bearing use.

CARE AND CLEANING INSTRUCTIONS:

- Do NOT use abrasive cleaners or sharp utensils to clean this product.
- Store the item in a cool, dry place.
- Wipe with a clean dry cloth.

DISPOSAL

We use recyclable or recycled packaging where possible


Please dispose of all packaging in accordance with your local authority's recycling regulations

At the end of the product's lifespan check with your local council authorized household waste recycling centre for disposal.

Plastic Polybags – this is made from the following recyclable plastic

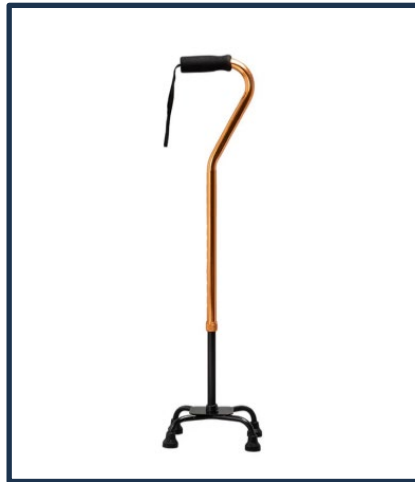Code & Symbol	
Type of Plastic	Low Density Polyethylene

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The Enterprise Department Ltd, Trading as Coopers of Stortford.