Product name: Bathtub grab bar
Item No: 696560 v001: 11/04/2025

3PAGEN

GB INSTRUCTION MANUAL

Please read and retain these instructions for future reference


GB - INSTRUCTION MANUAL

Product description with safety information

- Provides support to assist the user getting in and out of the bath.
- Rail handle is angled over the bath.
- Easily fixes to the bath with rubber-lined clamps to prevent marking.
- Handle height adjustable between: 62.5 - 82.5 cm (24.5 - 32.5 inch).
- Suitable for use with most baths, base plate fixes to floor.

DISPOSAL



- 3PAGEN use recyclable or recycled packaging where possible.
- Please dispose of all packaging, paper, cartons in accordance with your local authority recycling regulations.
- At the end of the product's lifespan please check with your local council authorised household waste recycling centre for disposal.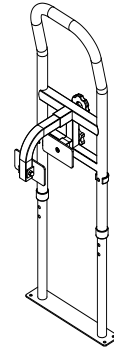

Assembly Instruction

Part List

①		X1
②		X1
③		X1
④		X1
⑤		X2
⑥		X4
⑦		X4

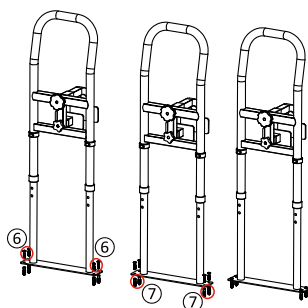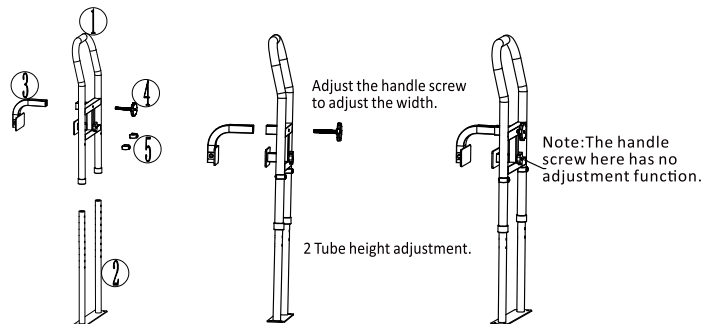After assembling the entire product, fix the iron plate at the bottom of the product to the ground outside the bathtub. Mark the four holes of the iron plate on the ground, each with a diameter of 6 mm and a depth of 30mm, and then knock the four ⑥ plastic plugs into the holes. Then align the holes of the iron plate with the holes on the ground, and use a hexagonal wrench to insert the ⑦ self-tapping screws through the iron plate into the plastic plugs. Tighten the screws until the iron plate is fixed, and then tighten all four holes.

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