GB-Stair assist walking stick


Cane Assembling

Assembling the handle and the main body:

1. Align the spring buttons on both sides of the handle A with the round holes of the main body B interface and insert.
2. After pressing the handle spring button and aligning it with the round hole of the body, the spring button pops up and is fixed to complete the assembly.

Assembling the cane legs and main body:

1. Align the cane legs C and D with the interface on the connection side of body B and insert.
2. Align the button on the cane's foot with the round hole in the main body and press the button.
3. Move the cane leg upward so that the button aligns with the round hole in the main body and then pops up and is fixed. The lower part is assembled.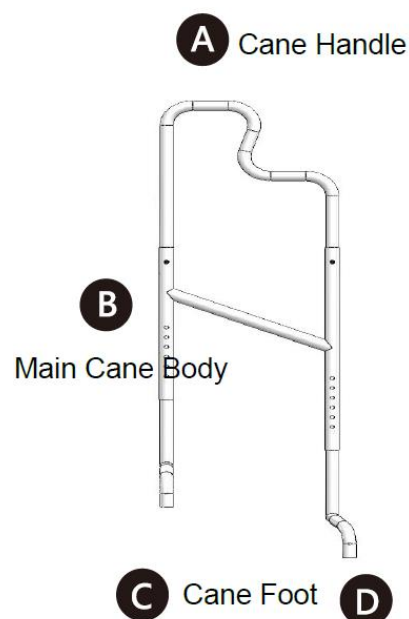

Adjustments before use


The height of the stair assist cane is $(\text{user height}/2) + 2\text{cm}$ = the suitable height. The posture for operating the crutch is to place the crutch 20cm in front of the body and the elbow angle is 30-40 degrees.

Correspondence user height and suitable cane height

User Height	150	152	154	156	158	160	162	164
Cane Height	77	78	79	80	81	82	83	84


User Height	166	168	170	172	174	176	178	180
Cane Height	85	86	87	88	89	90	91	92

User Height	182	184	186	188	190	192	194
Cane Height	93	94	95	96	97	98	99

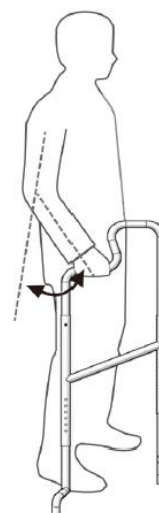
The way to go upstairs/downstairs

1. Place the cane above the steps you want to walk on. First confirm whether the height of the cane is appropriate, and then confirm whether the four legs of the cane are stable.
2. Hold the handrail of the stairs firmly with one hand, and hold the cane handle in the circle with the other hand.
3. Once your feet are firmly planted and the cane support your body and the ground stably, you can walk up/down stairs stably.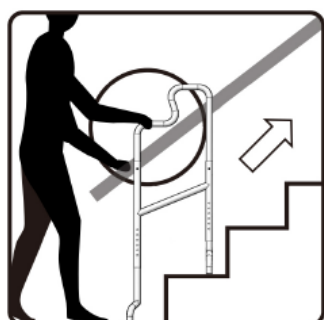


Height adjustment method

1. Press the button indicated by the arrow above and move the cane legs inward or outward.
2. Move to the required height, align the spring button with the round hole and pop it up to fix it.
3. Finally, tighten the lower locking collar to complete the assembly.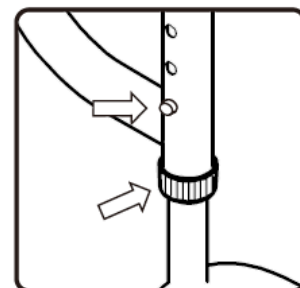


Adjust the cane

1. Place the stair assist cane on the stairs, first press the spring button on the low cane foot side to adjust the appropriate operating height, then press the high cane foot side spring button to adjust the height to fit the stair.
2. After the cane is adjusted to the desired height, align the spring button with the round hole on the cane foot and pop it out to fix it.
3. Hold the cane and move it to see if the cane foot is stably supported on the step. If it is stable and does not shake, tighten the locking collar and the adjustment is complete.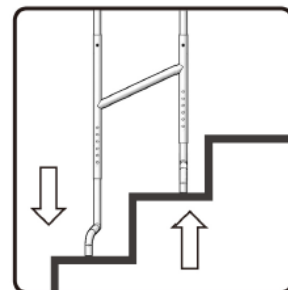

DISPOSAL

- use recyclable or recycled packaging where possible.
- Please dispose of all packaging, paper, cartons in accordance with your local authority recycling regulations.
- At the end of the product's lifespan please check with your local council authorised household waste recycling centre for disposal.

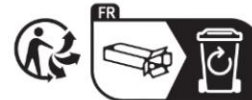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The Enterprise Department Ltd, Trading as Coopers of Stortford.
The Enterprise Department Ltd, Trading as -