Vacuum Ear Cleaners

Please read and retain these instructions for future reference

vivadia

Dimensions

(W) 10 x (D) 6 x (L) 13cm

Material: Plastic

- GENTLE EAR CLEANERS
- SAFELY REMOVE EAR WAX
- COMPLETE WITH THREE WASHABLE SILICONE TIPS, A FILTER AND A CLEANING BRUSH , USE IT TIME AND TIME AGAIN.

Batteries required: 2x AA (not included) (per unit)

ALWAYS USE ALKALINE BATTERIES.

DO NOT USE LITHIUM BATTERIES.

ASSEMBLY INSTRUCTIONS / INSTRUCTIONS FOR USE:


Unpack the components carefully. Please check all parts have been removed from the packaging.

WARNINGS

- Check with your paediatrician before using on young children
- Never use without the silicone attachments
- Wash thoroughly before and after each use

User Instructions:

1. Insert 2x AA batteries (not included) as shown:
2. Wash your hands thoroughly
3. Slide a silicone attachment (larger diameter in first) onto the small tube located in the front of the ear cleaner. There are four different colour silicone attachments for multiple users
4. Slide the ON/OFF switch to the ON position
5. Insert the tube with the silicone attachment into your ear, slightly turn the ear cleaner to help draw out the wax .

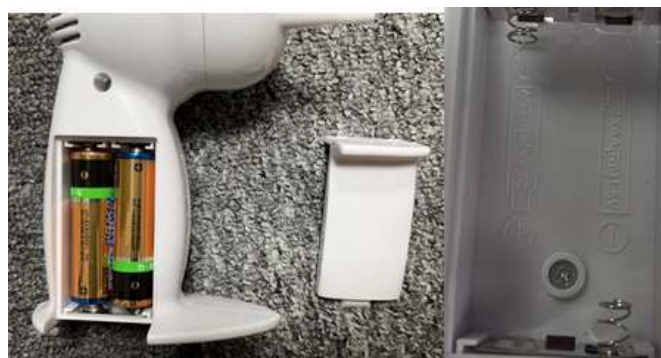BATTERIES

- Keep batteries away from children and pets.
- Do not mix used and new batteries.
- Remove the batteries if left for long periods.
- Never dispose of batteries in a fire.
- Battery disposal, spent or expired batteries must be properly disposed of and recycled in compliance with local regulations. For detailed information, contact your local authority.
- Follow the battery manufacturer's safety, usage, and disposal instructions.

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UK: The Enterprise Department Ltd.